

## 4 CONFIGURATION

This section describes the many aspects of XO FLEX that can be customized to fit your personal needs.

### 4.1 CONFIGURATION OF PATIENT CHAIR POSITIONS

1. Adjust the chair to the required position using the foot control joystick as described in section 3.5.2
2. Press the configuration button – see Figure 65 – within 8 seconds
3. The display will show “CHAIR”
4. Activate the chair base joystick – see Figure 66 – according to the position being configured:
  - : Working position 1
  - ⬅: Working position 2
  - ▼: Rinse position
  - ▽ (twice within one second): Entry position
5. Subsequently, an “OK” sound is heard
6. The display will show the configured position shortly afterwards



Figure 65 – Configuration button (1) under the instrument bridge

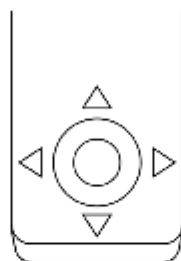


Figure 66 – Chair base joystick

### 4.2 CONFIGURATION OF GENERAL UNIT AND PATIENT CHAIR FUNCTIONS

1. Activate the configuration button (1) while all instruments are resting – see Figure 65.
2. Activate ▲ / ▼ or ↓ – see Figure 67 – to browse the parameter to change (see Table 5)
3. Activate ► to select the parameter to change
4. Use ▲ / ▼ to change the value of the selected parameter or ↓ to increase the value
5. Activate ◀ to save the new value