XP4

**Manuale di istruzioni per l'uso e la manutenzione • Operating instruction
Manuel d'utilisation et d'entretien • Manual de instrucciones y mantenimiento del usuario
Bedienungs- und Wartungsanleitung**

Dental unit type:
Riunito modello:

Serial number:
Matricola:

1ª edizione • Aprile 2016
1st edition • April 2016
1^{ème} édition • Avril 2016
1ª edición • Abril 2016
1. Auslage • April 2016

Cod. D4260450

▲ Dati tecnici • Technical data _____	4
Données techniques • Datos técnicos • Technische Daten _____	5
▲ Condizioni di trasporto e conservazione • Transportation and storage conditions _____	6
Conditions de transport et stockage • Condiciones de transporte y almacenamiento • Transport- und Lagerungsbedingungen _____	7
▲ Condizioni di utilizzo • Operating conditions _____	6
Conditions d'utilisation • Condiciones de uso • Nutzungsbedingungen _____	7
▲ Dispositivi di controllo del riunito • Control devices of the dental unit _____	6
Dispositifs de contrôle de l'équipement • Dispositivos de control del equipo dental • Steuerbefehle der Behandlungseinheit _____	7
___ Tastiera operatore • Dentist's control panel _____	6
Panneau de commande opérateur • Teclado del operador • Zahnarzt-Bedienfeld _____	7
___ Tastiera assistente • Assistant's control panel _____	8
Panneau de commande assistant • Teclado del asistente • Assistenten-Bedienfeld _____	9
▲ Uso del riunito • Use of the unit _____	8
Utilisation de l'équipement • Uso del equipo dental • Gebrauch der Behandlungseinheit _____	9
___ Accensione dell'apparecchio • Unit main switch _____	8
Allumage de l'appareil • Encendido del aparato • Einheit einschalten _____	9
___ Fusibili di protezione • Protection fuse _____	8
Fusibles de protection • Fusibles de protección • Schutzsicherungen _____	9
___ Posizionamento del paziente • Programming of the patient chair _____	8
Positionnement du patient • Colocación del paciente • Sitzpositionen des Patienten _____	9
___ Memorizzazione dei programmi • Memorization of the programs _____	10
Mémorisation des programmes • Memorización de los programas • Programme Speicherung _____	11
___ Riempimento del bicchiere • Cup filling _____	12
Remplissage du verre • Llenado del vaso • Becher füllen _____	13
___ Lavaggio bacinella • Bowl washing _____	12
Lavage crachoir • Lavado de la cubeta • Schalenspülung _____	13
___ Accensione lampada • Operating lamp's switch on/off _____	12
Allumage scialytique • Encendido de la lámpara • Lampe einschalten _____	13
___ Campanello • Key for door opener _____	12
Sonnette • Timbre • Ruftaste _____	13
___ Serbatoio alimentazione strumenti (Opzionale) • Physiological solution (Optional) _____	14
Réservoir alimentation instruments (En option) • Depósito alimentación instrumentos (opcional)	
Tank zur Instrumentenversorgung (Optional) _____	15
___ Pulsanti ausiliari • Auxiliary push buttons _____	14
Touches auxiliaires • Botones auxiliares • Zusatztasten _____	15
___ Pedaliera • Multifunctional foot control _____	14
Pédale • Pedal • Fußanlasser _____	15
___ Sistema di gestione spray • Spray management system _____	14
Système de gestion des sprays • Sistema de gestión de los sprays • Anwendung der Sprühfunktion _____	15
___ Funzioni programmabili degli strumenti • Programmable functions of the instruments _____	16
Programmes automatiques des instruments • Funciones programables de los instrumentos	
Programmierbare Funktionen der Instrumente _____	17

▲ Strumenti (parti applicate) • Instruments (applied parts) _____	18
Instruments (appliqués) • Instrumentos (partes aplicadas) • Instrumente (angewandte Komponenten) _____	19
___ Regolazione della pressione agli strumenti • The pressure regulation of the instruments _____	18
___ Réglage de la pression aux instruments • Regulación de la presión en los instrumentos • Druckregelung der Instrumente _____	19
___ Siringa operatore e siringa assistente • Dentist's syringe and assistant syringe _____	18
Seringue opérateur et seringue assistant • Jeringa del operador y jeringa del asistente	
Zahnarzt- und Helferinnen-Luft-/Wasserbläser _____	19
___ Turbina • Turbine _____	20
Turbine • Turbina • Turbine _____	21
___ Micromotore • Micromotor _____	20
Micromoteur électrique • Micromotor eléctrico • Elektro-Mikromotor _____	21
___ Ablatore • Ultrasonic Scaler _____	22
Détartreur • Ultrasonidos • Ablationsgerät _____	23
___ Polimerizzatore • Curing Light _____	22
Lampe à composites • Lámpara de polimerización • Polymerisationslampe _____	23
___ Lampada per illuminazione del cavo orale • Dental light _____	22
Lampe éclairage cavité buccale • Lámparas para iluminación cavidad bucal • Lampe zur Ausleuchtung der Mundhöhle _____	23
▲ Uso della poltrona • Use of the patient chair _____	22
Utilisation du fauteuil • Uso del sillón • Gebrauch des Behandlungsstuhls _____	23
___ Movimentazione del poggiatesta • Movements of the headrest _____	24
Déplacement de la tête • Regulación del reposacabezas • Kopfstütze verstellen _____	25
___ Movimentazione del bracciolo destro • Right armrest movement _____	24
Déplacement de l'accoudoir droit • Regulación del brazo derecho • Rechte Armlehne verstellen _____	25
▲ Manutenzione ordinaria • Ordinary maintenance _____	26
Entretien ordinaire • Mantenimiento ordinario • Ordentliche Wartung _____	27
___ Lavaggio del filtro aspirazione • Cleaning the suction filter _____	26
Lavage du filtre aspiration • Lavado filtro de aspiración • Absaugfilter reinigen _____	27
___ Lavaggio del filtro scarico bacinella • Cleaning of the cuspidor's drainage filter _____	26
Lavage du filtre vanne de crachoir • Lavado filtro de descarga de la escupidera • Abflussfilter der Speischale reinigen _____	27
___ Serbatoio alimentazione spray • The alimentation of the spray bottle _____	26
Reservoir extra-spray • Depósito alimentación spray • Externe Spray-Wasserversorgung _____	27
___ Separatore di amalgama • Mix separator _____	28
Séparateur récupérateur d'amalgame • Separador de amalgama • Amalgamabscheider _____	29
▲ Pulizia e disinfezione del dispositivo • Cleaning and disinfection of the unit _____	28
Nettoyage et désinfection du dispositif • Limpieza y desinfección del dispositivo • Reinigung und Desinfektion der Einheit _____	29
___ Pulizia delle superfici • Surface cleaning _____	28
Nettoyage des surfaces • Limpieza de las superficies • Reinigung der Oberflächen _____	29
___ Maniglie e protezione in silicone • Handles and protection in silicone _____	28
Poignées et protections en silicone • Manijas y protección de silicona • Griffe und Schutzvorrichtungen aus Silikon _____	29
___ Raccordi portacannule e tubi aspirazione • Cannulas and aspiration tubes _____	28
Raccords porte-canule, tubes d'aspiration • Racores portacánulas, tubos de aspiración • Anschlüsse, Kanülenhalterungen _____	29

___ Bacinella • Cuspidor bowl _____	30
Crachoir • Escupidera • Speischale _____	31
▲ Identificazione dell'apparecchio • Product Identification _____	30
Identification de l'appareil • Identificación del aparato • Gerätekenung _____	31
▲ Note tecniche • Technical notes _____	30
Remarques techniques • Notas técnicas • Technische Hinweise _____	31
▲ Smaltimento del dispositivo • Disposing of the device _____	32
Elimination du dispositif • Eliminación del dispositivo • Entsorgung _____	33
___ Definizione di RAEE • Definition of RAEE _____	32
Définition de DAEE • Definición de RAEE • Begriffsklärung RAEE _____	33
___ Modalità di smaltimento delle apparecchiature • Disposing modality of the equipment _____	32
Modalité d'élimination des équipements • Formas de eliminación de los aparatos • Geräteentsorgung _____	33
▲ Allegato I: tabelle previste dalla norma CEI EN 60601-1-2 (2010) _____	34
Enclosure I : Data according to EN 60601-1-2 (2010) _____	36
Annexe I: tableaux prévus par la norme CEI EN 60601-1-2 (2010) _____	38
Anexo I: Tablas previstas por la norma CEI EN 60601-1-2 (2010) _____	40
Anlage I: Tabellen gemäß Norm CEI EN 60601-1-2 (2010) _____	42

Dati tecnici

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Technical data

Trade mark: Swident
 Model: XP4
 Rated voltage: 230 V
 (~ alternated)
 Rated frequency: 50 Hz
 Rated power: 300 VA
 Product category: Class I Type B
 equipment

Classification according to EN 60 601-1.

5.3 Common equipments (Equipment with cover NOT protected from the water penetration);

5.5 The Equipments (units) are not to be operated in areas with risk of explosion; and closed to any apparatus producing electromagnetic emissions;

5.6 Equipments with continuous operation with intermittent load (=DAB).

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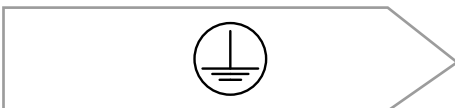
The equipment is conformed to EEC directive 93/42/CEE "Medical Devices"

The manufacturer accepts the responsibility for safety properties of our equipment only if:

- Installation, maintenance, repair and modifications are carried out by a person explicitly authorized by us;
- The general and electric installation of the room is conformed to the related laws;
- The equipment has been correctly set up according to our "operating instructions";
- The equipment has been correctly used according to our "User Manual".

The manufacturer declares that the equipment is a complex dental unit for being used on the man only by a high qualified practical staff in dental practice as job for therapy.

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Warning: any modification to this equipment is prohibited

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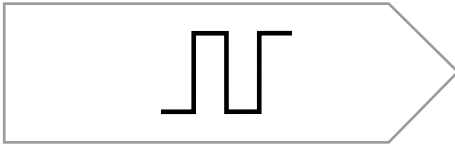
Disposal conforming to the RAEE standard

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Equipment for intermittent operation (chair: 2 minutes on - 18 minutes off)

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Transportation and storage conditions

When the equipment is packed for shipment and/or storage, it can support the following conditions:

- Environmental temperature from -40°C to +70°C;
- it should not be exposed to rain or excessive humidity.

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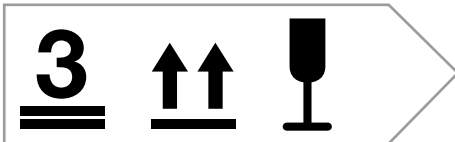
The other symbols on packaging are:

- maximum the number of packages stackable (3)
- the high side of the package
- the fragility of the content.

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Temperatura ambientale da +10°C a +35°C.

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Pressione atmosferica: da 700 a 1600 mbar.

Operating conditions

Environmental temperature from +10°C to +35°C;

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Control devices of the dental unit

Dentist's control panel

- 1 - Auxiliary button
- 2 - Patient chair, up / Patient chair program n°1
- 3 - Patient chair, down / Patient chair program n°2
- 4 - Backrest down / Patient chair program n°3
- 5 - Backrest up / Bowl position, last position
- 6 - Physiological solution to the instruments
- 7 - Predisposition of F.O. / Switch on/off operating lamp
- 8 - Reverse rotation of micromotor / Calling bell
- 9 - Increasing instrument's power / Cup filler
- 10 - Decreasing of instrument's power / Bowl flush
- 11 - Instrument/diagnostics maximum power indicator LED. The blinking LED indicates an anomaly: contact our technical support service.

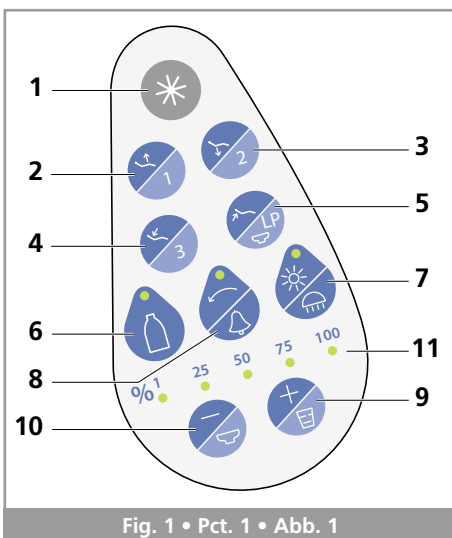
Fig. 1 • Pct. 1 • Abb. 1

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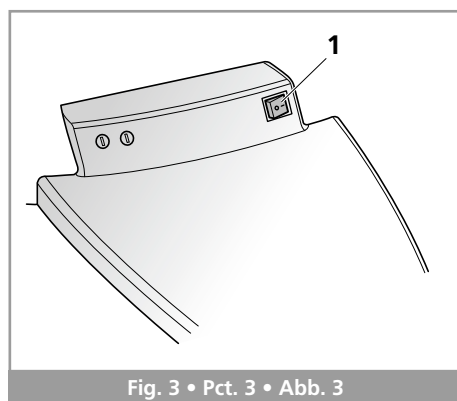
Assistant's Control panel

- 1 - Patient chair up / Patient chair program n°1
- 2 - Patient chair down / Patient chair program n° 2
- 3 - Backrest down / Patient chair program n° 3
- 4 - Back rest up / Bowl position, last position
- 5 - Switch on/off of operating lamp
- 6 - Bowl flush
- 7 - Cup filler

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
Use of the unit

Unit main switch

The unit gets ready for the operating pressing the main switch (position 1). The main switch will be illuminated by a green light.

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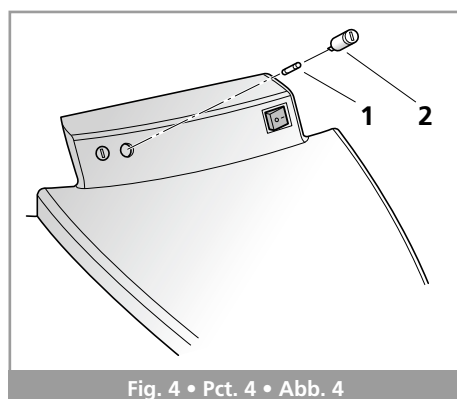Protection fuse

The dental unit is provided of 2 protection fuses for the power supply. To replace these fuses, you must unscrew the fuse holder (n° 2) and replace the fuse (n° 1). Use only fuses having the same characteristics of the original ones (T3,15A - 250V - 5x20).

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Programming of the patient chair positions

All keys control of the patient chair (table's dentist panel, table's assistant panel and foot control) have a double function: if they are pressed (>1 second), they start the driven movements; if they started by an impulse, they started the program.

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The longer pressing key activates the UP movement; with one short pressing key activates program patient chair no. 1.

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The longer pressing key activates the down movement; with one short pressing key activates program patient chair no. 2.

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The longer pressing key activates the inclination (down movement) of the backrest, with one short pressing key activates program patient no. 3.

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The longer pressing key activates the return (up movement) of the backrest. By one short pressing key activates the program that moves to rinsing position. Immediately after reaching this position, automatically the operating lamp is switch off and it's activated the function of the cup filler and then simultaneous the bowl flush function starts. By a second pressing key the patient chair returns to the last work position and then automatically the operating lamp is switched on.

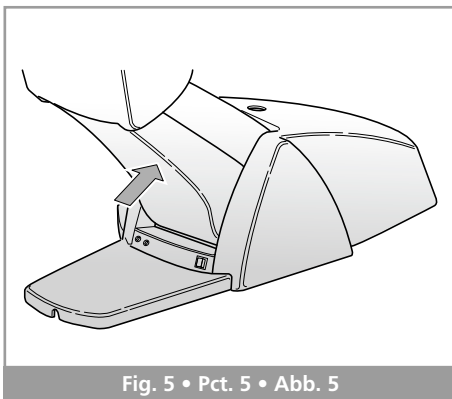
Fig. 5 • Pct. 5 • Abb. 5

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Memorization of programs

For changing one of the other four of the pre-selected functions stored (program no. 1, 2, 3 and LP), bring the patient chair into the desired position manually by pressing the program keys; then press towards the high the lower cover of the arm (Pantograph) of the patient chair and in the same time press the wanted program key and hold depressed until hearing a Beep sound, this is the confirmation of the new storage position instead of the old one.

The new stored program can now be called by pressing the appropriate key.

It is not allowed to move the patient chair when the instruments are taken from holders and are activated by the multifunctional foot control for the keys commands.

It is not allowed to move the patient chair when the cuspidor is turned towards the chair: this condition is signalled by Beep sound.

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Cup filling

The cup filing is manual automatic by factory setting; it's activated by pressing key no. 9 of the operator's control panel, only when the instruments are in their holders, or pressing key no. 6 of the assistant's control panel.

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The flushing time can be set adjusting the trimmer on the chair's PCB.

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Bowl Flushing

The flushing time is automatic factory setting; it's activated by pressing key no. 10 of the operator's control panel, only when the instruments are in their holders, or pressing key no. 7 of the assistant's control panel.

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Operating lamp's switch On/Off

The switch on of the operating lamp is activated by pressing key n°7 of the operator's control panel, only when the instruments are in their holders, or pressing key n° 5 of assistant's control panel. A second impulse gives the switch off.

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Key for door opener

Press key no. 8 of the operator's control panel, only when the instruments are in their holders its necessary an external circuit (max 24V-1A) for key for door opener or other services.

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Physiological solution (Optional)

If the bottle for physiological solution is present, you can select by the key board the liquid to the instrument's destination. By the switched on light situated to the upper part of the key board, the distribution of the liquid from the bottle should be made.

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The LED indicator must be switch off during the operating substitution of the bottle.

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Auxiliary push-buttons

The key n° 1 of the dentist's control panel is contact for auxiliary function, which activation has to be made by an authorized service technician.

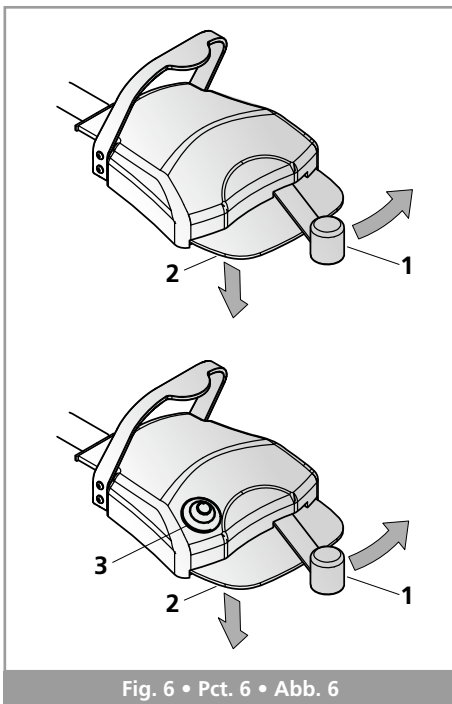
Fig. 6 • Pct. 6 • Abb. 6

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Multifunctional foot control

Pressing the foot lever 1 to the right side, any dynamic selected instrument is started. In case of instruments with a power or adjustable speed (as ultrasonic scaler or micro motor), the lever's opening angle is proportional to the power supplied to the maximum program key.

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Spray management system

Activation of the instrument spray can be configured using two different systems.

STANDARD SYSTEM: Turn lever 1 to turn the instrument on; press lever 2 simultaneously to activate the spray. If lever 2 alone is turned the chip is activated. No command or indication appears on the instruments table.

ALTERNATIVE SYSTEM: The spray must be selected from the instruments table using the "spray tank" button, which has a dual function:

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- With the instruments at rest the button controls and, when the LED is on, indicates insertion of sprays from the bottle.

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- With the instrument raised the button controls and, when the LED is on, indicates insertion of the spray in the raised instrument.

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The spray will be activated when lever 1 on the pedal is turned to start the instrument. If lever 2 is turned the chip is activated.

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Each instrument stores its last setting.

Configuration must be carried out by a qualified technician.

Joystick 3, situated on upper side of the foot control if present, controls the movements of the chair with the same modes as the control panels (see patient's position).

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Programmable functions of the instruments

The programming of the function's instruments has been made by the instrument taken from its holder. After returning the same instrument into the holder, all functions are automatically memorized.

When the instrument is taken from its holder, it's possible to set up all desired program functions of the same instrument.

By the key no. 7 of the dentist's control panel the instrument's F.O. is switched on or switched off; the L.E.D situated at the upper part of the key board shows the predisposition.

The F.O. will be switched on when the foot control will be driven and it will be switched off after 10 seconds from the activation of the same or immediately after returning the instrument into its holder.

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By the key n° 8 of the assistant's control panel, it is switched on or switched off the reverse operation of the micro motor; the switched on LED indicator situated on the upper part of the key board indicates the predisposition of the reverse operation.

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By the key no. 9 and 10 of the dentist's control panel, it's activated the program power or the maximum speed that will be had on the raised instrument, with a foot pedal in activation of the maximum intensity. Such power or speed can be selected on 5 levels: 1% - 25% - 50% - 75% - 100% and will be indicated from the relative LED indicator. Pressing the key + it's possible to increase the speed, while pressing the key - it is possible decrease the speed.

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The precision of the 5 power levels is functional to the performance of the instruments.

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Instruments (applied parts)

Following instruments are compatible with XP4 unit:

- Turbine: all types, if provided with Midwest connection and CE-marked.
- Micromotor: Bien Air, models MC2 LK/Led, MC3 LK/Led, MCX Led, TKD Definitive, air motors.
- Scaler: of brand Satelec model Newtron and Newtron Led, brand TKD mod. Titanus or brand EMS model Piezo Scaling, No Pain and No pain Led.
- Curing light: of brand Satelec model Mini LED, brand TKD model MOON.

The manufacturer supplies these instruments with original instructions for use. Follow the instructions in the documentation supplied with the devices for technical characteristics, performances, maintenance procedures and for proper use.

The pressure regulation of the instruments

Every instrument has an individual regulation of the spray; in the lower part of the dentist's instrument table are placed the regulation taps with the following functions:

- 1 - Screw tap: it regulates the volume of air or the cooling. This adjustment must be made by a qualified technician;
- 2 - Tap with Knurl: it regulates the water spray;
- 3 - Screw tap: it regulates the air of chip blower and spray. This adjustment must be made by a qualified technician.

Dentist's syringe and assistant syringe

This instrument does not have the program functions.

- 1 - Control Key for providing water;
 - 2 - Control Key for providing air;
- Pressing both keys (1 and 2) at the same time it will provide spray.

The syringe's cover can be unthread for permitting the sterilization, pressing the realising push button 3.

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Turbine

When the instrument is taken from its holder, the LED indicates the function's power percentage, automatically it will be appeared as 100% value and it cannot be changed. If the turbine has F.O., it's switched ON (and OFF, resp.) by pressing key board (LED indicator illuminated). The instrument has 3 pressure's regulators situated in the lower part of the lower side of the instrument's table. The turbines and Micromotors compatible with the unit are those ones arranged with Midwest connectors and with exclusively CE mark.

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Micromotor

When the instrument is taken from its holder, the same is arranged for the operation. The speed regulation depends on the pressure exercised on the foot control and the key of selection of the spin sense will be active. If the Fibre optic is arranged it will be active the key for switch it on / off. The instrument has the regulator of the spray placed in the lower part of the dentist's instrument table.

The air motors supplied with the unit are equipped by the standard connection type "E", conformed with a directive ISO 3964. It's expected the utilisation of the hand pieces exclusively CE marked.

Electrostatic discharges on the foot control base, can create the stop of the micromotor; in this case it is possible to restore the normal function resetting to zero the foot control (bringing lever 1 of the foot control completely to the left side).

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Ultrasonic Scaler

When the instrument is taken from its holder, the LED indicates the previously memorized power and it can be changed. If the Fibre optic is arranged it will be active the key for switch it on / off.

The instrument has one pressure's regulator for cooling water in the lower part of the doctor's table.

The oscillation of the ultrasonic scaler to the high frequency can compromise (interfered) the working of the pace-maker.

Curing Light

The curing light is always fed and ready-to-use.

For the application of others hand pieces, after the first unit's supply, please contact Swident.

Dental light

The following Faro dental light models can be installed on the unit:

- EDI halogen light
- MAIA LED light
- ALYA LED light

Refer to the original use and maintenance manuals supplied with the devices.

Use of the patient chair

Electrostatic discharges on the foot control base, can create the stop of the patient chair; in this case it is possible to restore the normal function resetting to zero the foot control (bringing the lever 1 of the foot control to the left side completely).

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Movements of the headrest

The headrest can be more or less extracted from the backrest only by pulling or pressing it because it is automatically blocked by a clutch. The double joint for wide articulation can be adjusted by the knob (1):

- In the case of the **standard** head (Fig. 11) rest turned in anticlockwise sense, it will be unblocked; in clockwise sense it will be blocked;

- In case of optional version pneumatic **"Bios"** (Fig. 12); maintaining pressed the push button, the head rest is unblocked; leaving the push button the head rest is blocked.

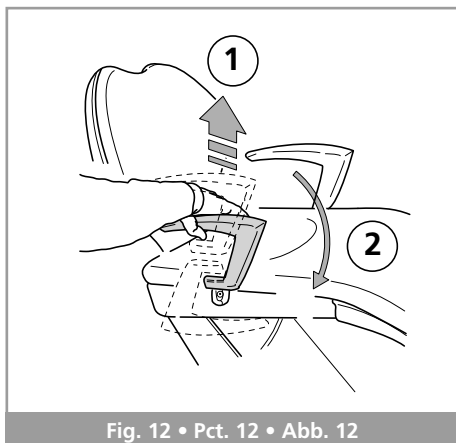
Fig. 12 • Pct. 12 • Abb. 12

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Right armrest movement

The right armrest can be lowered towards the front side chair to facilitate the patient's access. For doing it, first of all pull it up (1) and then turn it forwards and downwards (2). For restoring it in the normal position make the opposite operation

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Ordinary maintenance

Cleaning the suction filter

The aspiration should be cleaned daily. To take it out pull outside the yellow cover and detach the filler making pressure on the snap lock. The suction filter must be replaced on a weekly basis.

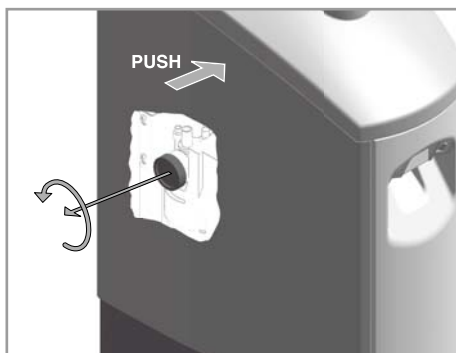
Fig. 14 • Pct. 14 • Abb. 14

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Cleaning of the cuspidor's drainage filter

The filter of the cuspidor has to be cleaned daily. To pull it unscrew cap in clockwise sense and unthread it. If for the alimentation of the unit it is used the demineralized water; it is suggested to put inside the filter a mineralized tablet that guarantees automatically the good operation of the aspiration devices.

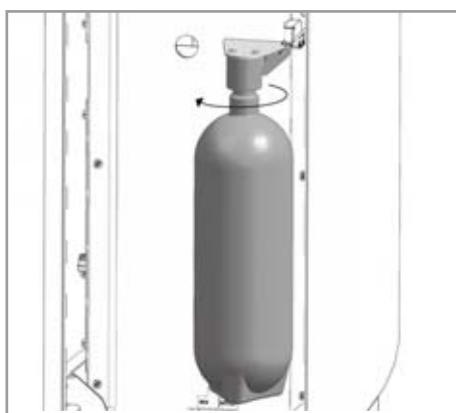
Fig. 15 • Pct. 15 • Abb. 15

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The alimentation of the spray bottle

Before taking off the clean bottle, be sure that the LED indicator situated at the upper part of the key board no. 6 of the dentist's instrument table has to be switched off, this guarantees that inside the bottle there is no pressure. For taking off the bottle, turn it in anticlockwise sense. To reinstall it repeat the inverse operation. Bottle operating pressure: 2.5 bar.

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WARNING: do not fill the bottle with saline solution. Fill the bottle only with water and, if necessary, disinfectant.

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For ensuring the safety of the device, it is recommended to replace the bottle every 5 years.

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For keeping the safety of the device, it is recommended to execute, every one (1) year, the tests of conformity to the electrical safety.

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Mix separator

In case a mix separator is installed, the device will signal the fill-up of the tank by a Beep sound.

The manufacturer supplies this instrument with original instructions for use. Follow the instructions in the documentation supplied with the device for technical characteristics, performances, maintenance procedures and for proper use.

If present, the Swipure filter must be replaced on a yearly basis. contact our technical support service.

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Cleaning and disinfection of the unit

Surface cleaning

It is advised to clean up the external surfaces of the unit with hygienic liquid for a plastic material.

The instruments tubes must be washed with water and neutral soap, then well dry.

The upholstery of the patient chair and the stools have to be cleaned up with water and neutral soap.

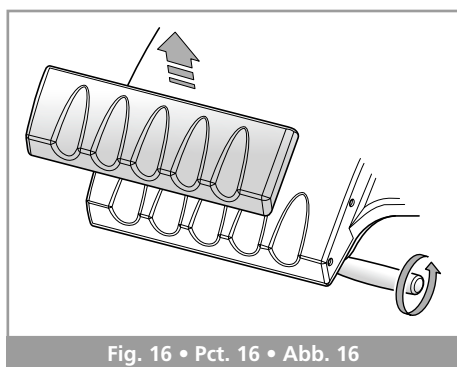
Fig. 16 • Pct. 16 • Abb. 16

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Handles and protection in silicone

The handles of the dentist's and assistant instrument table are taken off turning in clockwise sense and are cleaned by a disinfectant liquid.

The protection in silicone of the instruments table can be taken off for a better cleaning and disinfection using a disinfectant liquid.

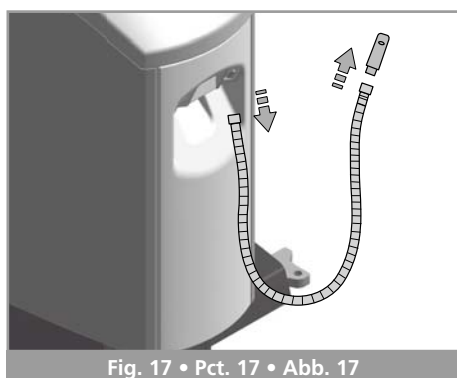
Fig. 17 • Pct. 17 • Abb. 17

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Cannulas and aspiration tubes

The connectors and the aspiration tubes can be taken off for cleaning up and disinfecting, from the relative connectors.

The cleaning of the cannulas must be done by disinfectant liquid.

The suction tubes must be replaced every year.

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Cuspidor bowl

The cuspidor and the nozzles of the cup filler can be removed for cleaning and disinfection pulling them. The cleaning of the small items must be done by a disinfectant liquid. Use vaseline oil to lubricate the O-rings in the nozzle connector.

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Product Identification

For any communication with the manufacturer or after sales centres, it must be always mentioned the equipment's model (mod.) and the serial number printed on the matriculation label.

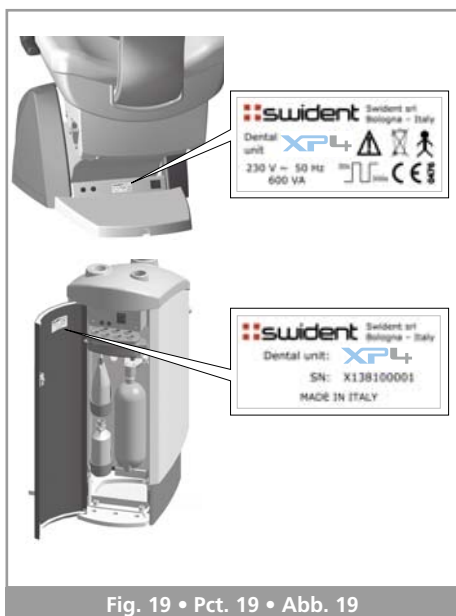
Fig. 19 • Pct. 19 • Abb. 19

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Technical notes

The wiring diagrams and the documents for assistance and maintenance are given available in the qualified technical central services. The technical assistance on the equipment must be done exclusively by a high qualified technician.

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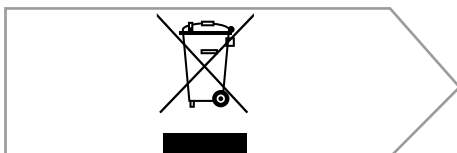
Disposing of the device

Definition of RAEE

The symbol "REEE" means Refusal of Electrical and Electronically Equipment; it has to be considered as RAEE all electrical and electronically equipments for domestic or professional use that at the end of their life must be "scraped".

Disposing modality of the equipment

At the end of the usefull life, expected by the manufacturer in 10 years from the installation date, the equipment must be made unusable following this instructions:

- Untie the electrical, hydraulic and pneumatic connections of the equipment;
- Cut all electrical connections which go from the base of the equipment to different peripherals;
- Cut the cables of the secondary exit of the transformer as they cannot be restored.

When the equipment is unusefull, it has to be packed by recyclable material and as to be decided from the customer or the retailer, it has to be sent to private or public rubbish dumps equipped for the disposing of the REEE. The customer or the retailer must take trace of the made "Disposing" of the equipment through the suitable structures, keeping the transport documents with precise motive's indication.

Proper separate waste collection and compliance with the instructions above help prevent adverse effects on human health and the environment and promote the re-use and/or recycling of the materials with which the equipment is made. Unlawful disposal of the product results in the application of the administrative sanctions provided by the laws in force.

Attention!! The symbol placed on the equipment, does not permit the disposing of the same with different ways from what we explained above.

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Enclosure I : Data according to EN 60601-1-2 (2010)

XP4 Dental Unit has been manufactured for use in an environment like the one cited below. The customer or user of XP4 should ensure that it is used in the correct environment.

Transmission Tests	Conformance	Electromagnetic environment
RF transmission	Group 1	XP4 use RF energy for its internal operation. Its transmission is very low and it is highly probable that it doesn't disturb neighbouring electronic devices.
RF transmission	Class b	XP4 is for use in all premises including residential ones and all premises that are, directly connected to a public low voltage power supply covering residential buildings.
Harmonic transmission	Class A	The dental unit must emit electromagnetic energy to carry out its intended function. This may affect electronic devices located near the unit.
Transmission of voltage Fluctuations or Flicker	Conforms	

Immunity test	EC 60601 test level	Conformance level	Electromagnetic environment
Electrostatic discharge	± 6 kV contact discharge ± 8 kV atmospheric discharge	± 6 kV contact discharge ± 8 kV atmospheric discharge	Floors made of wood or concrete or have ceramic tiles. If they are covered with synthetic material, the relative humidity must be at least 30%
Fast transient electrical disturbances / lines	± 2 kV for power lines ± 1 kV for input and output	± 2 kV for power lines ± 1 kV for input and output	The quality of the supply should correspond to that of a typical business or hospital environment
Surges	± 2 kV for power lines ± 1 kV for input and output	± 2 kV for power lines ± 1 kV for input and output	The quality of the supply should correspond to that of a typical business or residential environment
Voltage interruptions, short-term interruptions and fluctuations of the supply voltage	<5% Ut (95% interruption) for 0,5 period	<5% Ut (95% interruption) for 0,5 period	The quality of the supply should correspond to that of a typical business or residential environment. If the user of XP4 asks continued operation even if the power supply is interrupted, it is recommended to supply XP4 Dental Unit from an uninterrupted power supply.
	40% Ut (60% interruption) for 5 periods	40% Ut (60% interruption) for 5 periods	
	70% Ut (30% interruption) for 25 periods	70% Ut (30% interruption) for 25 periods	
	<5% Ut (95% interruption) for 5 seconds	<5% Ut (95% interruption) for 5 seconds	
Magnetic field with a supply (50Hz)	3 A/m	3 A/m	Magnetic fields at the mains frequency should correspond to typical values in a business and residential environment.

Immunity test	EC 60601 test level	Conformance level	Electromagnetic environment
Conducted HF disturbances	3 Veff	3 Veff	$d = 1,2 \sqrt{P}$ with P as the maximum rated power of the transmitter in Watts (W) according to the transmitter manufacturer and d as the recommended safe distance in meters
Radiated HF disturbances	3 V/m	3 V/m	$d = 1,2 \sqrt{P}$ for 80 Mhz to 800 Mhz $d = 2,3 \sqrt{P}$ for 800 Mhz to 2,5 Ghz with P as the maximum rated power of the transmitter in Watts (W) according to the transmitter manufacturer and d as the recommended safe distance in meters

XP4 is intended for use in an electromagnetic environment in which HF disturbances are controlled. The user of XP4 can help prevent electromagnetic disturbances by maintaining the minimum distance between portable and mobile HF telecommunications devices (transmitters) and the XP4 depending on the output of the communication device as indicated. Do not use the dental unit near or stacked with other equipment. Should it be necessary to use it near other equipment, be sure to monitor its normal operation.

Rated power of the transmitter in W	Safe distance depending on the transmission frequency		
	150 KHz to 80 Mhz	80 Mhz to 800 Mhz	800 Mhz to 2.5 Ghz
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	12	2,3
10	3,8	3,8	7,3
100	12	12	23

Maximum length of power cords:

- Chair supply: 1.2 mt AC
- Foot control: 1.5 mt

WARNING: The use of accessories, transducers and cables other than those specified, except those sold by the manufacturer as spare parts, may cause increased emissions or decreased immunity of the device.

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Swident S.r.l.